



polaluminate

ENGLISH

Brush-on tooth whitening system (6% Hydrogen Peroxide) INSTRUCTIONS FOR USE

Pola Luminat is a pleasantly flavoured, 6% hydrogen peroxide whitening solution. Designed to easily brush directly onto teeth to form a thin whitening film without the need of trays. Pola Luminat can easily fit into your pocket or handbag for you to be able to use whenever convenient for you. Pola Luminat is a great way to maintain your Pola smile whenever and where ever you want. Before using Pola Luminat, your dentist should assess your suitability for the product including your medical history and explain the application procedure.

INDICATIONS FOR USE:

- Whitening of discolored teeth

INSTRUCTIONS:

1. Brush and floss teeth as normal.
2. Blot dry the front surface of teeth with a tissue.
3. Whilst retracting the lips, apply a thin layer of Pola Luminat solution to the front surfaces of the teeth, re-loading brush as required.
4. After application do not let your lips touch your teeth for 30 seconds to ensure a thin whitening film forms on the teeth.
5. Leave Pola Luminat solution on teeth for 30 minutes. Do not eat or drink during this time to maximize results.
6. Use Pola Luminat twice a day for best results.

For best results, one of the daily treatments should be applied before going to bed, allowing the Pola Luminat to stay on overnight. When not in use, **firmly** replace cap to prevent solution from drying out.

STORAGE AND HANDLING:

- Store in a cool and out of direct sunlight at temperatures between (2°-25°C / 35°-77°F).

PRECAUTIONS:

- Irritant. Avoid contact with eyes or gums. Avoid swallowing.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Not to be used by pregnant, lactating women or person under 18 years old.
- Avoid contact with eyes, rinse immediately if product comes into contact with them.
- Use at room temperature.
- Pola Luminat should not be used by patients with known allergies or chemical sensitivity to peroxides, glycerol etc.
- Do not smoke during or immediately after the whitening process.
- Keep product away from heat and naked flame, as product is flammable.
- Discard Pola Luminat after treatment has been completed.
- SDS available at www.sdi.com.au or contact your regional representative.
- Contains Sodium Fluoride and contains Hydrogen Peroxide.
- Do not use after expiry date.
- Keep out of reach of children and pets.

FIRST AID:

For advice, contact a Poisons Information Centre or a doctor.

- **Eye (contact):** Hold eyelids apart and flush with running water for at least 15 minutes. Seek medical advice if symptoms persist.
- **Skin (contact):** If skin or hair contact occurs, remove contaminated clothing and flush skin and hair with running water.
- **Ingestion:** If swallowed, do NOT induce vomiting. Drink plenty of milk or water and seek medical advice.
- **Inhalation:** No symptoms expected.

Made in Australia by SDI Limited
Bayswater, Victoria 3153
Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Ireland 00800 0225 5734
Italy 00800 0225 5734
New Zealand 0800 734 034
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166

www.sdi.com.au
www.polawhite.com.au

EC REP
SDI Germany GmbH
Hansestrasse 85
51149 Cologne
Germany
T+49 2203 9255 0

PAK41327 A

