



polapaint

E
N
G
L
I
S
H

Paint-on tooth whitening system

(8% carbamide peroxide)

INSTRUCTIONS FOR USE

Pola Paint is an easy to use paint-on tooth whitening gel. It is a pleasantly flavored gel with a low viscosity which is easily applied onto the tooth surface. Before using Pola Paint, your dentist should assess your suitability for the product including your medical history and explain the application procedure.

INDICATIONS FOR USE:

- Whitening of discolored vital teeth
- Whitening of discolored non-vital teeth

INGREDIENTS: Ethanol, Glycerin, PEG-135, Aqua, Carbamide Peroxide, Aroma, Tripropylamine, Carbomer, Sodium Fluoride, Limonene

NET WEIGHT: 5.1g

HYDROGEN PEROXIDE EQUIVALENT:

Pola Paint 8% Carbamide Peroxide = 2.8% Hydrogen Peroxide

INSTRUCTIONS:

1. Brush and floss teeth as normal.
2. Blot dry the front surface of teeth.
3. Whilst retracting the lips, apply a thin layer of Pola Paint gel to the front surfaces of the teeth, re-loading brush as required.
4. After application do not let your lips touch your teeth for 30 seconds.
5. Leave Pola Paint gel on teeth for 30 minutes. Do not eat or drink during this time to maximize results.
6. At the end of 30 minutes, remove the Pola Paint gel by brushing with a tooth brush.
7. Use Pola Paint twice a day for 14 days.

For best results, one of the daily treatments should be applied before going to bed, allowing the Pola Paint to stay on overnight. When not in use, **firmly** replace cap to prevent gel from drying out.

STORAGE AND HANDLING:

- Store in a cool and out of direct sunlight at temperatures between 4°C - 20°C / 39°F -68°F
- Do not store in clothing pockets.

PRECAUTIONS:

- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Not to be used by pregnant or lactating women or person under the age of 18 years.
- Use at room temperature.
- Pola paint should not be used by patients with known allergies or chemical sensitivity to peroxides, glycerol etc.
- Do not smoke during or immediately after the whitening process.
- Keep product away from heat and naked flame, as product is flammable.
- Discard Pola Paint after treatment has been completed.
- MSDS available at www.sdi.com.au or contact your regional representative.
- Contains Sodium Fluoride and contains Hydrogen Peroxide.

FIRST AID:

- **Skin (contact):** Pola paint may be irritating to skin or soft tissue. If any irritation occurs, remove gel, rinse mouth area with plenty of water and drink a glass of milk. Discontinue treatment and consult your dentist if there is persisting sensitivity or any reaction to the gel.
- **Eyes (contact):** Open eye and flush thoroughly with water for 15 minutes. Seek medical attention.

Made in Australia by SDI Limited
 Bayswater, Victoria 3153
 Australia 1 800 337 003
 Austria 00800 022 55 734
 Brazil 0800 770 1735
 France 00800 022 55 734
 Germany 0800 100 5759
 Ireland 01 886 9577
 Italy 800 780625
 New Zealand 0800 734 034
 Spain 00800 022 55 734
 United Kingdom 00800 022 55 734
 USA & Canada 1 800 228 5166
www.sdi.com.au
www.polawhite.com.au

EC REP

SDI Dental Limited
 Block 8, St Johns Court
 Swords Road, Santry
 Dublin 9, Ireland
 Tel +353 1 886 9577

